PERSONAL CLEANLINESS AND LENS HANDLING

PREPARING THE LENS FOR WEARING

- Always wash your hands thoroughly with a mild soap, rinse completely, and dry with a lint-free towel before touching your lenses.
- Avoid the use of soaps containing cold cream, lotion, or oily cosmetics before handling your lenses, since these substances may come into contact with the lenses and interfere with successful wearing.
- Handle your lenses with your fingertips and be careful to avoid contact with fingernails. It is helpful to keep your fingernails short and smooth.

HANDLING THE LENSES

- Always insert and remove the same lens first to avoid mix-ups.
- Position the lens on your index finger and examine the lens to be sure that it is moist, clean, clear, and free of any nicks or tears. Make sure the lens is the right way out.

- Should you accidentally place an inside-out lens on your eye, one of the following signs should signal you to remove and replace it correctly:
 - Less than usual comfort
 - The lens may fold on the eye
 - Excessive lens movement on blink
 - Blurred vision
- If the lens folds and sticks together, place the lens in the palm of your hand and wet thoroughly with the rewetting solution recommended by your ECP. Then GENTLY rub the lens between your index finger and palm in a back and forth motion.
- If the lens flattens or drapes across your finger, the lens or your finger may be too wet. To correct this, dry your finger by transferring the lens several times from one index finger to the other, drying the opposite finger each time.

PLACING THE LENS ON THE EYE

The One Hand Placement Technique (FIG. A and FIG. B)

Place the lens on your index finger. With your head up, looking straight ahead, pull down your lower eyelid with the middle finger of your placement hand. Look up steadily at a point above you. Then place the lens on the lower white part of your eye. Remove your index finger and slowly release the lower eyelid. Look down to position the lens properly. Close your eyes for a moment and the lens should center itself on your eye.

The Two Hand Placement Technique (FIG. C and FIG. D)

With the lens on your index finger, use the middle finger of the other hand to pull the upper eyelid against the brow. Use the middle finger of your placement hand to pull down the lower eyelid and then place the lens centrally on your eye. While holding this position, look downward to position the lens properly. Slowly release your eyelids.

FIG. A

FIG. B

FIG. C

FIG. D

If the lens feels uncomfortable, then:

Look in a mirror and gently place a finger on the edge of the contact lens and slowly slide the lens away from your nose while looking in the opposite direction. Then by blinking, the lens will recenter itself. If the lens still feels uncomfortable, follow the steps described in ADVERSE REACTIONS (PROBLEMS AND WHAT TO DO).

CENTERING THE LENS

If after placement of the lens your vision is blurred, check for the following:

- The lens is not centered on the eye. To center a lens, follow one of the procedures below:
 - Hold the upper and lower eyelids open with your fingers. Then, while looking in a mirror, gently place a finger on the contact lens and slide the lens towards the center of the eye.
- OR
 - Hold the upper and lower eyelids open with your fingers. Then, while looking in a mirror, move your eye towards the lens to place it on the center of the eye.
- If the lens is centered, remove the lens and check for the following:
 - Cosmetics or oils on the lens. Discard the lens and use another lens.
 - The lens is on the wrong eye.
 - The lens is inside-out (it would not be as comfortable as normal).

If you find that your vision is still blurred after checking the above, remove both lenses and consult your ECP.

REMOVING THE LENS

- Always remove the same lens first.
- Wash, rinse, and dry your hands thoroughly.
- Always be sure that the lens is in the correct position on your eye before you try to remove it (a simple check of your vision, closing one eye at a time, will tell you if the lens is in the correct position). Look up and slowly pull down your lower eyelid with the middle finger of your removal hand and place your index finger on the lower edge of the lens. Squeeze the lens lightly between the thumb and index finger and remove it. Avoid sticking the edges of the lens together.
- Remove the other lens by following the same procedure.
- Use rewetting drops recommended by your ECP if lens is difficult to remove.

Note: If this method of removing your lens is difficult for you, your ECP will provide you with an alternative method.

CARE FOR A STICKING (NON-MOVING) LENS

If a lens sticks (stops moving), put a few drops of the rewetting solution recommended by your ECP into your eye. DO NOT use plain water or anything other than the recommended solutions. DO NOT attempt to remove a lens that is sticking, which could damage your eye. If the lens does not begin to move when you blink after several applications of the solution, immediately contact your ECP. DO NOT attempt to remove the lens except on the advice of your ECP.

INFORMATION SPECIFIC TO MONOVISION OR MULTIFOCAL WEARER

- You should be aware that as with any type of lens correction, there are advantages and disadvantages to monovision or multifocal contact lens therapy. The benefit of clear near vision in straight ahead and upward gaze that is available may be accompanied by a vision compromise that may reduce your visual acuity and depth perception for distance and near tasks.
- Some patients had trouble adapting to monovision contact lens therapy. Symptoms, such as mild blurred vision, dizziness, headaches, and a feeling of slight imbalance, may last for a brief minute or for several weeks as adaptation takes place. The longer these symptoms persist, the poorer your prognosis for successful adaptation.
- You should avoid visually demanding situations during the initial adaptation period. It is recommended that you first wear these contact lenses in familiar situations that are not visually demanding. For example, it might be better to be a passenger rather than a driver of an automobile during the first few days of lens wear. It is recommended that you only drive with monovision or multifocal correction if you pass your driver's license requirements with monovision or multifocal correction.
- Some patients will never be fully comfortable functioning under low levels of illumination, such as driving at night. If this happens, you may want to discuss with your ECP having additional contact lenses prescribed so that both eyes are corrected for distance when sharp distance binocular vision is required.
- If you require very sharp near vision during prolonged close work, you may want to have additional contact lenses prescribed so that both eyes are corrected for near when sharp near binocular vision is required.
- Some patients require supplemental spectacles to wear over the monovision or multifocal correction to provide the clearest vision for critical tasks. You should discuss this option with your ECP.
- It is important that you follow your ECP's suggestions for adaptation to monovision or multifocal contact lens therapy. You should discuss any concerns that you may have during and after the adaptation period.
- The decision to be fitted with a monovision or multifocal correction is most appropriately left to the ECP in conjunction with you, after carefully considering and discussing your needs.**

EMERGENCIES

If chemicals or hazardous substances of any kind (e.g., household products, gardening solutions, laboratory chemicals, etc.) are splashed into your eyes, you should: FLUSH EYES IMMEDIATELY WITH TAP WATER AND THEN REMOVE LENSES PROMPTLY. CONTACT YOUR ECP OR VISIT A HOSPITAL EMERGENCY ROOM WITHOUT DELAY.

REPORTING OF SERIOUS INCIDENTS

Any side effects, adverse reactions, or product complaints from use of Bausch + Lomb contact lenses should be reported to the manufacturer. Country specific information can be found on www.bausch.com/contactus. Reporting of serious incidents from use of Bausch + Lomb contact lenses should be reported to the manufacturer and the relevant competent authority.

HOW BAUSCH + LOMB SOFT CONTACT LENSES ARE SUPPLIED

Each lens is provided in a foil sealed plastic container containing packaging solution. The contact lenses are supplied steam sterilized.

STORAGE CONDITIONS

Reference the temperature symbol below for the temperature storage conditions, if applicable.

The products below must not be stored below 1°C (34°F):

SofLens® daily disposable (hilafilcon B) Visibility Tinted Contact Lenses
SofLens® daily disposable Toric (hilafilcon B) Visibility Tinted Contact Lenses
Bausch + Lomb Naturelle™ daily disposable (hilafilcon B) Cosmetically Tinted Contact Lenses
Biotrue® ONEday (nesofilcon A) Visibility Tinted Contact Lenses
Biotrue® ONEday For Astigmatism (nesofilcon A) Visibility Tinted Contact Lenses
Biotrue® ONEday For Presbyopia (nesofilcon A) Visibility Tinted Contact Lenses

Bausch + Lomb Incorporated
1400 North Goodman Street
Rochester, NY 14609 USA

Bausch + Lomb Incorporated
Cork Road Industrial Estate
Waterford, X91 V383, Ireland

EU IMPORTER

Bausch + Lomb Netherlands B.V.
Koolhovenlaan 110, 1119 NH Schiphol-Rijk
P.O. Box 75774, 1118 ZX Schiphol
The Netherlands

SYMBOLS & ABBREVIATIONS GLOSSARY

The following may appear on the label or the carton of your product:

Medical device

Fee paid for waste management

Caution: Federal (USA) law restricts this device to sale by or on the order of a licensed healthcare practitioner

Sterilized using steam

Do not use if package is damaged and consult instructions for use

Do not re-use

Consult instructions for use (Explanation of symbols is also available at www.bausch.com/symbols)

Caution

DIA Ø_i

Diameter

BC

Base curve

PWR F_v

Power

AX

Cylinder axis

SPH

Sphere power

CYL

Cylinder power

ADD

Add power

©/TM are trademarks of Bausch + Lomb Incorporated or its affiliates.
© 2022 Bausch + Lomb Incorporated or its affiliates

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